



Getting started.

EY-LO runs most easily on the **iPad** — straight from the App Store. It is also available for **Mac** and **Windows**; the matching installers are included on this USB stick.

RECOMMENDED · IPAD WITH M-CHIP

- 01** Open the **App Store**, search for “**EY-LO**” and download the app.
- 02** Plug the camera cable into the iPad’s **USB-C port**.
- 03** Open EY-LO — done. No account, no installation, no drivers.

Continue at “Mount the camera on your head” on the next page. The iPad route is the fastest and most robust option.

ALTERNATIVE · LAPTOP (ONLY IF NEEDED)

MAC

Open the **Mac App Store**, search for “**EY-LO**” and download.

Already bought it for iPad? Then it’s **free** here (one purchase covers iPad + Mac).

App Store not working? Open the **EY-LO file (.dmg)** on this stick and drag **EY-LO** into the **Applications** folder.

On first launch, **allow camera access**.

WINDOWS

Double-click **EY-LO-Setup.exe** on this stick and click through the installer.

If SmartScreen warns: **More info** → **Run anyway**.


On first launch, **allow camera access**.



Camera & cable.

MOUNT THE CAMERA ON YOUR HEAD

- 01 Leave a small **service loop** coming from the **camera arm** — it gives freedom of movement and relieves strain on the connector.
- 02 Route the cable along the **headband**, over your head to the back.
- 03 Feed it out through the **holder on the headband** at the back (see sketch).
- 04 Put on the headband, adjust fit and tilt.
- 05 Attach the **cable clip** to your clothing (strain relief).

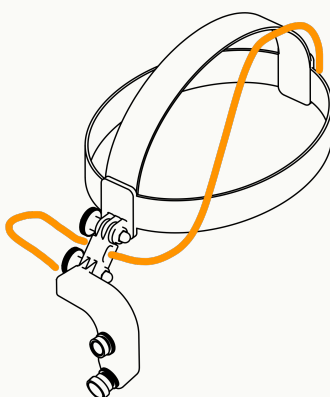

Cable routing: service loop at the front, then over the headband to the back.

CONNECTING

- 01 Connect the camera cable — iPad: **USB-C**, laptop: **USB**.
- 02 Start EY-LO — **Wide** and **Macro** are detected automatically.
- 03 If Wide/Macro are swapped: top right → **Settings** → swap cameras.

ALIGNING THE CAMERA

- 01 On the patient, look through your loupes at a **single tooth**.
- 02 Position the **Macro camera (lower one)** as close as possible and parallel to the loupe oculars.
- 03 Adjust so the **target tooth sits centered** in the image.
- 04 Set the **focus** of the Macro camera to your working distance by **turning the lens**.



Get going.

RECORDING

- 01** Start a **recording** or **snapshot** right in the app.
- 02** For a better overview, extend your screen to a larger monitor via **AirPlay** or an **HDMI adapter**.

Cameras swapped or want to change the recording folder? It's all top right under **Settings**. You can also restart the initial setup there at any time.

HELP & SUPPORT

Questions or problems? We're happy to help:
support@ey-lo.com